Highly Filled Flowable Composite

For Professional Dental Use Only

Product Description

OT Flow HF is a next-generation **highly filled flowable composite** that combines the strength and durability of a conventional composite with the superior adaptability of a flowable resin. With an increased filler load, it provides exceptional wear resistance, compressive and flexural strength, and reduced shrinkage, making it ideal both as a liner/base and for direct restorations in stress-bearing areas. Its thixotropic viscosity ensures easy handling, precise placement, and optimal adaptation to cavity walls, delivering long-term reliability and esthetic results in both anterior and posterior applications.

Composition

Bis-GMA (Bisphenol A-glycidyl methacrylate)
UDMA (Urethane dimethacrylate)
TEGDMA (Triethylene glycol dimethacrylate)
Barium–alumino–borosilicate glass and colloidal silica fillers (~78 wt%)
Photoinitiators (camphorquinone, EDMAB)
Pigments and stabilizers

Indications for Use

Class I–III restorations.
Cervical and root surface lesions.
Small posterior restorations and minimally invasive cavities.
Base or liner under direct composite restorations.
Resin coating and pediatric restorations.

Contraindications

Known hypersensitivity to methacrylate-based resins.
Direct pulp exposure.
Do not use in patients with known resin allergies.

Warnings

Avoid contact with eyes, skin, and mucosa. In case of contact, rinse immediately with water.
Uncured material may cause irritation or allergic reaction.
Use rubber dam isolation where possible.
For professional dental use only.

Precautions

Ensure the cavity is clean, disinfected, and dry before application.
Apply in ≤ 2 mm increments to ensure full light-curing.
Do not use eugenol-containing materials prior to application.
Use a curing light with an intensity ≥ 1000 mW/cm² and verify regularly.
Replace syringe cap immediately after use to avoid premature curing.

Properties (Typical)

Filler content: ~78 % by weight

Working time: Unlimited (until light exposure)

Curing time: 20 seconds (2 mm layer)

Compressive strength: > 350 MPa

Flexural strength: > 130 MPa

Shrinkage: < 2 %

Radiopacity: ≥ 3 mm Al

Shade: Universal tooth color

Viscosity: Thixotropic, easy flow under pressure

Directions for Use

Prepare and isolate the tooth according to standard protocols.

Apply the bonding agent and air dry according to manufacturer's instructions.

Dispense OT Flow HF directly into the cavity using a dispensing tip.

Adapt with a probe or brush to cavity walls.

Light-cure each increment (≤ 2 mm) for 20 seconds using a curing light ≥ 1000 mW/cm².

Finish and polish using conventional composite finishing systems.

Storage and Handling

Store at 10–25 °C in a cool, dry place.

Avoid exposure to sunlight and heat.

Do not freeze.

Keep syringe capped tightly when not in use.

Shelf life: 24 months from manufacturing date (see package).

Adverse Reactions

In rare cases, methacrylate components may cause allergic reactions.

Discontinue use and seek medical attention if redness, swelling, or irritation occurs.

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