**OT Light-Cure Calcium Silicate Liner
For Professional Dental Use Only**

Product Description

OT Light-Cure Calcium Silicate Liner is an advanced bioactive liner designed to protect the pulp and promote dentin regeneration in deep cavity preparations. Utilizing calcium silicate technology with high alkalinity, ion release, and bioactivity, it provides a protective barrier that supports pulp vitality. Its light-curing formulation ensures immediate setting, easy handling, and long-term stability. Suitable for use under composites, amalgam, and glass ionomer restorations, it creates a thin, uniform layer with excellent sealing and mechanical properties.

Composition

Tricalcium silicate
Dicalcium silicate
Calcium hydroxide
Strontium glass
Silicon dioxide
Photoinitiators (camphorquinone, EDMAB)
Methacrylate-based resin matrix
Pigments and stabilizers

Indications for Use

Indirect and direct pulp capping.
Protective liner under composite, amalgam, or glass ionomer restorations.
Base material for deep cavity preparations and pulp protection.

Contraindications

Do not use in patients with known allergies to methacrylates or calcium silicate components.
Not indicated for direct pulp exposures with excessive bleeding or infection.
Avoid contact with soft tissues.

Warnings

Avoid contact with eyes, skin, and oral mucosa. If contact occurs, rinse immediately with water.
Uncured material may cause irritation or sensitivity.
Use under adequate isolation to prevent contamination.
For dental professional use only.

Precautions

Ensure the cavity is clean, disinfected, and lightly dried before application.
Avoid desiccating dentin or exposing to strong air drying.
Use in thin layers (≤ 1 mm) to ensure complete light polymerization.
Do not use eugenol-containing materials in combination, as they may inhibit polymerization.
Replace cap immediately after use to prevent premature curing.

Properties (Typical)

Setting mechanism: Light-cure
Working time: Unlimited (until light activation)
Curing time: 20 seconds with halogen or LED curing light
Film thickness: ≤ 50 μ m
Radiopacity: ≥ 3 mm Al
Alkaline pH after curing: $\approx 9-10$
Compressive strength: >100 MPa
Color: Tooth shade (off-white)

Directions for Use

Prepare the cavity and isolate the working area.
Clean the dentin surface and remove debris.
Dispense a small amount of material onto the cavity floor using a suitable applicator.
Apply a thin, uniform layer (≤ 1 mm).
Light-cure for 20 seconds using a curing light with at least 1000 mW/cm² intensity.
Proceed with the placement of the restorative material after curing.

Storage and Handling

Store at 10–25 °C in a cool, dry place.
Avoid direct sunlight and high temperatures.
Do not freeze.
Keep tightly capped when not in use.
Shelf life: 24 months from manufacturing date (see package).

Adverse Reactions

Rare allergic reactions to methacrylate components may occur.
Discontinue use if redness, swelling, or irritation develops and seek medical advice.

Packaging

- Two (2) $\times$ 2 g syringe
- Applicator tips (single-use)
- Instruction leaflet (this IFU)

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